# RS Model II

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en

#### Description:

RS Model II is a light-curing resin for the additive manufacturing of high-precise dental models. The material has been optimized for Rapidshape® DLP® D Series systems with a wavelength of 385/405 nm for layer thicknesses of 50/100 µm.

#### Composition:

Preparation made of acrylates and initiators.

#### Application:

Additive manufacturing of dental models.

#### Use:

##### BUILD PROCESS

↘ **Prepare data set:** The orientation and type of support structures can be found in the system supplier's instructions for use.

↘ **Establish variable process parameters:** The appropriate process parameters are selected according to the specifications of the printer system supplier. Ensure that the correct material-specific set of parameters is selected (product, layer thickness, exposure time).

↘ **Mixing materials:** Stir material to the bottom of the bottle.

First filling: Pour material into the material reservoir of the unit.

Refill: Refill the material as needed and mix in the reservoir with a silicone spatula. Allow the reservoir with material to rest in the unit for 10 min.

↘ **Start build process.**

##### CLEANING AND DRYING

↘ **Clean part:** Separate the parts (with support structures) from the build platform with a spatula. The component is cleaned under flowing isopropanol for max. 5 min. If necessary, carefully clean the objects with a brush and a spray bottle filled with isopropanol.

↘ **Dry part:** Blow component with compressed air and leave to dry for 30 min at room temperature or in a drying cabinet adjusted to a temperature of approx. 37°C.

##### POST-CURING

↘ For post-curing, the components have to be cleaned and cooled to room temperature.

Curing Device	Post-curing
RS Cure	Parameter set <i>RS Model II</i> 6 min / vacuum 50 mbar / power 100 %
RS Cure XL	Parameter set <i>RS Model II</i> 6 min / vacuum 10 mbar / power 100 %

The device used must be maintained and calibrated according to the manufacturer's specifications to ensure mechanical properties and color.

↘ Remove support structure if necessary.

↘ For post-curing without oxygen connection, clean under flowing conditions with isopropanol for 3-5 min. If necessary, carefully clean the objects with a brush and a spray bottle filled with isopropanol.

↘ Blow component with compressed air and leave to dry.

##### ↘ POST-PROCESSING COMPLETED

#### Further processing instructions:

Parts are subsequently processed and finalized using standard dental techniques.

#### Warnings:

May cause an allergic skin reaction. Causes skin irritation. Avoid contact with unpolymerized material. Wear protective gloves, protective clothing, eye protection. If on skin: Wash with plenty of water and soap. If skin irritation or rash occurs: Get medical advice/ attention. Avoid breathing dust/ vapours/ spray. Switch on extraction system. The material is harmful to the environment. Avoid release to the environment. Collect spillage. Please always observe the respective valid safety data sheets. Always observe the relevant country-specific health and safety regulations and legal requirements when handling hazardous substances.

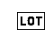
#### Storage instructions / shelf life:

Protect from exposure to light. May polymerize spontaneously. Store between 4°C and 25°C. Close after use. Do not use after the expiration date. See imprint/label for expiration date.

#### Disposal instructions:

Dispose of contents/container in accordance with local regulations.

The product was developed for use in dentistry and must be used according to the instructions for use.

	Batch number		Keep away from light
	Item number		Temperature limit
	Manufacturer		Observe directions for use
	Use-by date		Stir before use
	Exclamation mark		Environmental Hazard
	Distributor		Manufacturing Date

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